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The screenshot shows the main interface of the intelligent medical imaging cloud platform. On the left, there is a sidebar with icons for various services: 放射云 (Radiology Cloud), 本院检查 (In-hospital Exams), 远程诊断 (Remote Diagnosis), 超声云 (Ultrasound Cloud), 病理云 (Pathology Cloud), 内镜云 (Endoscopy Cloud), 心电云 (ECG Cloud), 眼科云 (Ophthalmology Cloud), 远程会诊 (Teleconsultation), 远程宣房 (Remote宣房), 远程示教 (Remote Teaching), 数据统计 (Data Statistics), and 质控管理 (Quality Control Management). The current page is '远程诊断'.

The main content area is titled '本院检查' (In-hospital Exams) and shows a table of examination results. The table has columns: 全部检查 (All Exams), 待检查 (Pending), 待诊断 (Pending Diagnosis), 诊断中 (In Progress), 已诊断 (Diagnosed), 患者ID (Patient ID), 姓名 (Name), 性别 (Gender), 年龄 (Age), 检查ID (Check ID), 检查状态 (Check Status), 设备类型 (Device Type), 检查项目 (Check Item), 检查类别 (Check Category), 检查时间 (Check Time), 打印时间 (Print Time), 登记日期 (Registration Date), 归档状态 (Archiving Status), and 操作 (Operations). There are 792 records in total, with page navigation buttons at the bottom.

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The screenshot shows the 'Create Consultation' interface. At the top, there's a header with icons for back, forward, and search. Below it is a section titled '填写会议详情' (Fill in meeting details). This section contains fields for '会议号' (Meeting ID), '会议链接' (Meeting link), '会诊主题' (Consultation topic), and '会诊时间' (Consultation time) with a date range from July 3, 2023, to July 3, 2023, and times from 16:30 to 17:30. There's also a '自动识别' (Automatic recognition) button with a placeholder '请在此粘贴腾讯会议分享信息，点击识别后自动填充左侧相关信息'. A '提交申请' (Submit application) button is at the top right. Below this, there's a '邀请对象' (Invited objects) field with a '仅选择科室' (Only choose department) checkbox. Under '会诊类型' (Consultation type), '单学科会诊' (Single specialty consultation) is selected. A '理由/要求' (Reason/Requirement) text area has a character limit of 1000. The bottom section is for '患者信息' (Patient information) with fields for '关联患者' (Associated patient) via HIS号 (HIS number), '患者姓名' (Patient name), '患者ID' (Patient ID), 'HIS号' (HIS number), '患者性别' (Patient gender) with options '未知' (Unknown), '男' (Male), and '女' (Female), '证件号' (Document number) for ID card, '出生日期' (Date of birth), and '手机号' (Mobile phone number).

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The screenshot shows the 'Create Consultation' page. At the top, there is a text input field labeled 'Please paste information' with placeholder text 'Please paste meeting sharing information, click recognition after identification to automatically fill in the corresponding information on the left'. Below this are two buttons: 'Operation Guide >' and 'Clear' (清空). A blue button labeled 'Recognition' (识别) is highlighted. The main form area has the following fields:

- Consultation Information:** 'Meeting ID' (会议号) is required and empty. Placeholder: 'Please enter the meeting ID or recognize and fill in'. 'Meeting Link' (会议链接) is required and empty. Placeholder: 'Please enter the meeting link or recognize and fill in'. 'Consultation Topic' (会诊主题) is required and empty. Placeholder: 'Please enter the consultation topic or recognize and fill in'.
- Consultation Start Time:** 'Consultation Start Time' (会诊开始时间) is required and set to 'Today Tuesday 11:00'.
- Consultation End Time:** 'Consultation End Time' (会诊结束时间) is required and set to 'Today Tuesday 12:00'.
- Invited Objects:** 'Invited Objects' (邀请对象) is required and empty. Sub-options include 'Add 0 experts' (已添加0位专家), 'Select Department' (仅选科室), and 'Select Expert' (选择专家).
- Consultation Type:** 'Consultation Type' (会诊类型) is required and set to 'Single specialty consultation' (单学科会诊).
- Reason/Requirement:** 'Reason/Requirement' (理由/要求) is required and empty. Placeholder: 'Please enter the reason and requirements for the consultation, no more than 1000 characters'.

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©2013–2026 腾讯云版权所有